

HOSPITAL CARE

Versatus™

I.V. Catheter

Product Information



**Hard case
primary packaging**

The Versatus™ I.V. Catheters are made out of medical grade polyurethane which becomes soft at body temperature ensuring patient comfort.

Description	Raw Materials
1 Catheter	Polyurethane + BaSO ₄
2 Catheter hub Caulking pin	Polypropylene + master batch Stainless steel
3 Cannula (internal needle)	Stainless steel
4 Needle hub	Polycarbonate
5 Filter	Polyester – chlorinated PVC
6 Protector cap	Polypropylene + master batch
7 Protector	Polypropylene

Does not contain components made of natural rubber latex.

Sterilization

With ethylene oxide to a Sterility Assurance Level of at least 10⁻⁶

Storage

- Shelf life of 5 years
- The batch number and expiry date are printed on the label of each unit pack
- Do not store at extreme temperature and humidity

Your catheters of choice for a wide range of I.V. applications

Product Range

Product Code	Colour	External Catheter Ø		Internal Catheter Ø	Catheter Length	Flow Rate ml/min.
SR*OF2419	Yellow	24G	0.68 mm	0.47 mm	19 mm	15
SR*OF2225	Blue	22G	0.85 mm	0.60 mm	25 mm	38
SR*OF2051	Pink	20G	1.10 mm	0.80 mm	51 mm	59
SR*OF2032	Pink	20G	1.10 mm	0.80 mm	32 mm	67
SR*OF1851	Green	18G	1.30 mm	0.95 mm	51 mm	88
SR*OF1832	Green	18G	1.30 mm	0.95 mm	32 mm	99
SR*OF1651	Grey	16G	1.70 mm	1.30 mm	51 mm	184
SR*OF1464	Orange	14G	2.17 mm	1.73 mm	64 mm	282

CE 0197 STERILE EO

Regulatory Status

CE Certification – Class IIa medical device

Notified body

TÜV Rheinland 0197

Manufacturer

TERUMO CORPORATION
2-44-1 Hatagaya, Shibuya-Ku, Tokyo 151-0072, Japan

EC Representative

TERUMO EUROPE N.V. Interleuvenlaan 40, 3001 Leuven, Belgium

Packaging

Unit Pack Hard Case

Unit Box 50 pieces

Shipping Carton

200 pieces

Gross Weight 10.8 kg

Dimensions

266 x 690 x 263 mm

Precautions

- Do not use if unit package is damaged
- Do not re-use
- Do not attempt to re-insert a partially or completely withdrawn needle
- Dispose of safely after single use to avoid risk of infection

Always consult the product label for a complete overview of warnings,cautions and/or precautions prior to actual use.

This range complements the wider Terumo product offering including a comprehensive portfolio of both safety and conventional I.V. products which can be found at our web page on www.terumo-europe.com

Your Terumo partner



TERUMO EUROPE N.V.

Terumo Corporation Terumo Europe N.V.
+81 3 6742 8500 +32 16 38 12 11

www.terumo-europe.com

All brand names are trademarks or registered trademarks of TERUMO CORPORATION and their respective owners. Published by Terumo Europe N.V.